



Preparation of W&H surgical straight/contra-angle

handpieces and surgical motors

Notes on preparation

and surgical motors in particular.

These hygiene and care recommendations apply for W&H straight/contra-angle handpieces



135°C

Transmission instruments and dental motors labelled with this symbol are approved for the sterilization procedure recommended.



with this symbol can be cleaned and disinfected in a thermo washer disinfector.

Use certified and tested disinfectants for

not contain chlorine.

Use clean, dry drive air.

W&H transmission instruments and surgical

motors and observe the corresponding dosage and reaction time. The disinfectant used must

W&H recommends the use of demineralized water.Please see the detailed information at wh.com

Transmission instruments and surgical motors labelled

General notes

- Please note that transmission instruments must be cleaned and disinfected immediately after every treatment.
- The use of protective clothing (tear-resistant gloves, goggles, mouth protection) is recommended.
 Comply with the detailed information in the
- manufacturer's operating instructions.
- Preparation





2











3







adapters or disassemble them before placing them in the thermo

Lubrication

washer disinfector.



Manual

Machine













Packaging / Sterilization

135°C { } { }



sterilization stages!

W&H recommends sterilization

Please make sure that the load is dry after both cleaning and

according to EN 13060.



Dust-free, clean and dry.

Further information at wh.com/hygiene

office@wh.com wh.com